EN

Intended for use

Tooth protector. A dental splint intended to protect teeth from damages caused by instrument in connection with peroral endoscopy, intubation or heavily biting in connection with ECT. Used at hospitals by professional user.

Contraindications

Pediatric patients

Features

- CAMO disposable splints can be used both in the upper and lower jaws.
 - The size of the splint has been constructed to fit 90% of all jaw sizes. The size can, however, be easily adjusted with scissors.
 - Both jaws can be protected simultaneously without excessively diminishing the necessary area for peroral endoscopy.
 - The splints absorb high pressure without damaging teeth, thus the splints may be used during all kinds of pathological or induced excitations.
 - Biocompatible material
- In case of a serious incident from using the instruments, immediately contact Directa AB and the applicable authority in your country.

Precaution/ Warnings

- Do not reuse
- Shall not be sterilized or re-used as intended use can be jeopardized
- Film wrapping the filling shall not be removed as it prevents fragment dislodging.

Serious Incidents

In case of a serious incident from using the instruments, immediately contact Directa AB and the applicable authority in your country.

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Kontakt umiddelbart Directa AB og gjeldende myndighet i landet ved alvorlige hendelser med bruk av instrumentene.
**DIRECTA AB**