# ANA Etching Gel

Phosphoric Acid

**MAXI PACK**

**Contents**

10 Syringes à 2 ml

**DIRECTA**  
directadental.com

**DIRECTA AB**

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## Warning

H314 Causes severe skin burns and eye damage.

P280 Wear protective gloves/protective clothing /eye protection/face protection.

P305 + P351 + P338; IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.

P310 Immediately call a POISON CENTER or doctor/physician.

Contents ANA Etching Gel,  
Phosphoric acid 37%

CAS nr. 7664-38-2

## General Information

ANA Etching Gel is a 37% phosphoric acid gel which has been designed for an optimum level of fluidity and thixotropy. It can be applied very precise over the desired area.

## Contents

37% phosphoric acid in water, 4,7% Silicon dioxide.

## Indication

Etching of tooth substance using the enamel etch or total etch technique.

## Side Effects

None reported.

## PROCEDURE

1. Preparation of the cavity with minimal tooth reduction. For pulp protection, follow the instructions for material used.

### Selective Etch

2. Apply etching gel onto enamel only. Leave for 30 sec.
3. Rinse with water for 20 sec.
4. Dry with oil- and water-free air.

### Total Etch

2. First apply etching gel onto enamel only.
3. After 15-20 sec. brush it out in the whole cavity. Leave for 5-10 sec.
4. Rinse with water for 20 sec.
5. Dry with oil- and water-free air.

After etching the application of an adhesive, etc. can follow. Refer to separate instructions for use of the material used.

## Storage

Shelf life 3 years when from manufacturing date stored at < 25°C, see expiry date.

## Additional Notes

- Any strong acid like ANA Etching Gel can burn or irritate tissues. Be careful not to splash into eyes, onto mucous membranes, skin or any other areas. Confine to areas to be etched only.
- In case of contact with eyes rinse immediately with plenty of water and seek medical advice.
- In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible)
- Avoid contact with gingiva and adjacent teeth as well as longer contact with dentine. Etching gel may dissolve the liner.
- Avoid etching gel contact with oral soft tissues and skin. If accidental contact occurs, flush immediately with copious amounts of water. Refer to a physician if necessary.
- For dental use only.

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