## CALASEPT Temp

## Instructions for use

CALASEPT Temp is a proven, ready-to use close-up-material.

## It guarantees reliable sealing due to the following properties:

- rapid curing in the cavity;
- strong adhesion to dentin;
- outstanding marginal seal;
- resistant to chewing pressure

It damages neither the pulpa nor the gums.

## Composition:

Calcium sulphate-hemihydrate 50% Glycerine 20%, Zinc oxide 15%

## Type of application:

Use a filling instrument to apply the required quantity of CALASEPT Temp in the moist cavity. Curing begins after a few minutes. We recommend protecting the exposed location for 15 minutes. Deep cavities do not have to be completely filled, which makes subsequent removal easier. When sealing inlays, as far as is possible, ensure pressure-free application. A pointed, short and strong probe is best suited for removing the set of this material and insert it between dentin and filling.

## Side effects:

Undesirable side-effects are extremely rare if the product is prepared and used properly. Local dysaesthesia (e.g. taste irritations or irritations of the oral mucosa) can however, not be completely precluded. Should you become aware of any undesirable side effects, including in cases of doubt, please notify us, giving as precise a description of the attendant circumstances and symptoms as possible.

## Working instructions/precautionary measures:

Ensure that the working instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incident arise even though these instructions were observed, then please inform us and give a precise description of the accompanying circumstances.

## Storage/shelf-life:

Always close container immediately after removing material. Shelf-life of the closed packaging: See label. The preparation can be stored for at least 6 months after being opened for the first time. Do not store above 22°C. Do not use after expiry of the use-by date.

Distributed by:  
**DIRECTA AB**  
P.O. Box 723, Finvids väg 8  
SE-194 27 Upplands Väsby, Sweden  
Tel: +46 8 506 505 75  
www.directadental.com  
info@directadental.com

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**Willmann & Pein GmbH**