

## IPR System

**Solutions for most cases – The ContacEZ IPR Strip System** provides the dental professional with accurate, easy-to-use devices to meet most patient IPR needs.

**Contoured contacts** - The high quality stainless steel strips are flexible to preserve and conform to the natural contours of the teeth during IPR. They are also tight enough to reach even the tightest contacts with ease.

**Accurate** – Paired with the Incremental Thickness Gauges, ContacEZ provides systematic, precise enamel removal without cutting lips, gums or cheeks.



The **ContacEZ IPR System** is an innovative set of dental instruments specifically crafted to make interproximal reduction safe, fast, and easy. These flexible dental strips are ergonomically designed, patient-friendly and each strip is marked with the thickness of the strip and whether it is single-sided (S) or double-sided (D). ContacEZ's interproximal reduction strips make IPR safer and more accurate without creating sharp corners or cutting lips and gums. Designed for use with clear aligners and orthodontic treatments, these strips prevent excessive enamel removal and maximize patient comfort and safety. Autoclavable and reusable.

### IPR Strip System



#### Yellow Starter

0.07 mm  
Single-Sided. Extra-Fine Diamond. Serrated  
**Prod No:** 32108 8 pcs, 32132 32 pcs



#### Red Opener

0.12 mm  
Double-Sided. Medium Diamond  
**Prod No:** 32408 8 pcs, 32432 32 pcs



#### Blue Widener

0.15 mm  
Double-Sided. Coarse Diamond  
**Prod No:** 32508 8 pcs, 32532 32 pcs



#### Green Extra-Widener

0.20 mm  
Double-Sided. Coarse Diamond  
**Prod No:** 32608 8 pcs, 32632 32 pcs

### IPR PLUS Strip System



#### Clear Opener

0.10 mm  
Single-Sided. Med-Fine Diamond. Serrated  
**Prod No:** 32208 8 pcs, 32232 32 pcs



#### Cyan Opener

0.15 mm  
Single-Sided. Medium Coarse Diamond  
**Prod No:** 32308 8 pcs, 32332 32 pcs



#### Purple Widener

0.25 mm  
Double-Sided. Coarse Diamond  
**Prod No:** 32708 8 pcs, 32732 32 pcs



#### Brown Widener

0.30 mm  
Double-Sided. Coarse Diamond  
**Prod No:** 32808 8 pcs, 32832 32 pcs



#### Mint Widener

0.35 mm  
Double-Sided. Coarse Diamond  
**Prod No:** 36608 8 pcs, 36632 32 pcs



#### Aqua Widener

0.40 mm  
Double-Sided. Coarse Diamond  
**Prod No:** 32708 8 pcs, 32732 32 pcs



#### Turquoise Widener

0.45 mm  
Double-Sided. Coarse Diamond  
**Prod No:** 36808 8 pcs, 36832 32 pcs



#### Teal Widener

0.50 mm  
Double-Sided. Coarse Diamond  
**Prod No:** 32908 8 pcs, 32932 32 pcs

#### Prod No Product Description

32008 IPR Assorted (0.07mm - 0.20mm) Small, 2 Yellow, 2 Red, 2 Dark Blue, 2 Green. 8 pcs

32024 IPR Rainbow Set (0.07mm - 0.50mm) 3 Yellow, 3 Red, 3 Blue, 3 Green, 2 Clear, 2 Cyan, 2 Purple, 2 Brown, 1 Mint, 1 Aqua, 1 Turquoise, 1 Teal. 24 pcs

## Restorative Strip System

The three-step **ContactEZ Restorative Strip System** offers you an easy and effective way to finish a crown seating, breaking the process down into three simple steps: **Adjust**, **Clean**, and **Polish**. Autoclavable and reusable.



| Prod No | Product Description  |
|---------|--|
| 31008   | <b>Restorative Strip System Assorted.</b> 8 pcs<br>2 Black, 1 Orange, 1 Grey, 2 White, 2 Blue  |
| 31040   | <b>Restorative Strip System Assorted.</b> 40 pcs<br>8 Black, 8 Orange, 8 Grey, 8 White, 8 Blue   |
| 31023   | <b>Restorative Signature Set.</b> 24 pcs<br>1 Clear LAB, 1 Cyan LAB, 3 Black, 3 Orange, 2 Grey, 3 White, 3 Blue, 2 Gold, 2 Copper, 2 Blue Narrow, 2 Pink |

### Adjust




**Black** - Diamond Strip  
Single-Sided. Fine Diamond  
**Prod No:** 31108 8 pcs, 31132 32 pcs



**Orange** - Serrated Diamond Strip  
Single-Sided. Extra-Fine Diamond  
Double Serrated  
**Prod No:** 31208 8 pcs, 31232 32 pcs

### Clean

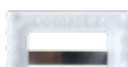

**White** - Gentle Saw  
Double Serrated. No Diamond  
**Prod No:** 31408 8 pcs, 31432 32 pcs



**Blue** - Heavy Duty Saw  
Double Serrated. No Diamond  
**Prod No:** 31508 8 pcs, 31532 32 pcs

### Polish



**Grey** - Final Polishing Strip  
Single-Sided. Ultra-Fine Diamond  
**Prod No:** 31308 8 pcs, 31332 32 pcs

## Subgingival Strip

The **ContactEZ Subgingival Strip** is designed to carefully and conveniently remove hard-to-reach subgingival rough margins and overhangs. They provide easy access to the subgingival area while avoiding damaging patient's gingival tissue and lips. In addition, these ultra-narrow strips are ideal for cleaning out cement debris in the subgingival area of the interproximal space after crown cementation. Autoclavable and reusable.



| Prod No | Product Description  |
|---------|--|
| 33108   | <b>ContactEZ Subgingival Strip System Assorted.</b> 8 pcs<br>2 Blue, 3 Gold, 3 Copper  |
| 33124   | <b>ContactEZ Subgingival Strip System Assorted.</b> 24 pcs<br>8 Blue, 8 Gold, 8 Copper |



**Gold**, 0.09 mm  
Single-Sided. Medium Diamond  
**Prod No:** 33408 8 pcs, 33432 32 pcs



**Blue**, 0.07 mm  
No Diamond. Double Serrated  
**Prod No:** 33708 8 pcs, 33732 32 pcs



**Copper**, 0.06 mm  
Single-Sided. Extra-Fine Diamond  
**Prod No:** 33608 8 pcs, 33632 32 pcs

## Hygienist Strip

Remove stains and clean proximal contact areas without damaging enamel surfaces. The **ContactEZ Hygienist Strip** is an advanced, high-precision dental strip designed to remove extrinsic discoloration and plaque from interproximal surfaces without causing damage to tooth enamel. The non-abrasive, flexible, stainless-steel strips clean proximal contact areas of tough stains from coffee, tea, tobacco, and red wine. It is quick, safe and ensure patient satisfaction and save hygienist chair time. Autoclavable and reusable.



**Pink**  
Perforated. Stainless Steel  
**Prod No:** 35108 8 pcs, 35132 32 pcs

35008 **ContactEZ Hygienist Removal Kit**  
2 Subgingival Blue, 2 Subgingival Copper,  
2 Subgingival Gold, 2 Hygienist Strips. 8 pcs

Video: ContactEZ -  
In the clinic with Cecilia Ömo

