CALASEPT® Plus

Calcium Hydroxide

Calcium hydroxide paste
Kalciumhydroxidpasta
Calciumhydroxid paste
Pâte d'hydroxyde de calcium
Pasta di idrossido di calcio
Kalsiumhydroksidi tahna
Kalsiumhydroksidipasta
Calciumhydroxidpasta
Pasta de hidróxido calcico
Pasta de hidróxido de cálcio
Calciumhydroxidepasta
Wodorotlenek wapnia w pascie
Pasta hydroxidu vápenatého
Kaltsiumhüdrosiid pasta
Kalcijev hidroksid pasta
Kalcium-hidroksid paszta
Kalcio hidroksido pasta
Pasta kalcijevega hidroksida
πάστα στείρου υδροξειδίου του ασβεστίου
Калциев хидроксид Паста
Kalcija hidroksīda pasta
Pastă de hidroxid de calciu
Pasta s hydroxidom vápenatým

CE
0413

DIRECTA AB

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Symboler

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Device description

Calcium hydroxide based endodontic dressing material used for endodontic and restorative treatment.

- Paste
- PH-value of 12.4.
- Pre-filled syringe of 1,5ml for multi-dosage.

Contents

Calcium hydroxide
Barium sulphate

REF

1230100 – 4x1,5ml
1230200 – 2x1,5ml

Indications for use

- Temporary root filling material.
- Direct pulp capping.
- Indirect pulp capping in deep cavities.

Contraindications

- Permanent root filling.
- Primary teeth.
- Patients with known sensitive to any of the ingredients.

Warnings

- Caustic to skin and mucous membranes.
- Caustic and can cause irritation on nerve tissue and necrosis if excess material distributed outside the foramen.

Pre-caution

- Use personal protective equipment to avoid contact with the skin, mucus membranes and eyes.
- Use rubber dam.
- Ensure the application needle is correctly attached to the syringe prior to use to avoid exposing the material in the oral cavity.
- Ensure correct application technique to avoid excess material distributed outside the foramen.
- After use and removal of application needle, re-seal the syringe with the protective cap. Ensure proper closing. All contact between calcium hydroxide paste and air will result in formation of calcium carbonate which will harden after some time.
- Improper storing can cause drying out or separating of the paste.
- Disinfect the exterior surfaces of the syringe with soaked wipes between uses to reduce the risk of cross-contamination.

Recommended storage conditions

- Keep the syringe tightly closed.
- Store at <25°C.
- Do not refrigerate.
- Avoid exposure to direct sunlight.

Shelf life

- 2 years from manufacturing date.
- Expiry date: see syringe and/or outer packaging.

Disposal

- Unused material shall be disposed according to information in safety data sheets.

Directions for use**General:**

Use standardized endodontic and/or restorative treatment protocols (clinical procedure)

Prior to use:

1. Remove the white cap. Extrude a small amount of Calasept paste and discard.
2. Attach a Calasept Plus Flexi-Tip to the syringe. Note: Calasept Plus Flexi-Tip is supplied non-sterile.
3. Eject a small amount of paste to equalize pressure in the syringe and to control flow of paste through the delivery device (Calasept Plus Flexi-Tip).

Temporary root filling:

- If Calasept is applied directly into the root canal, make sure to reduce minimum 2mm from WL (working length) with Calasept Plus Flexi-Tip and use a back fill technique.
- When applying paste gradually, intermittently compress with absorbing paper points or cotton pellets.
- Calasept can also be applied with a paste spiral (Lentulo).

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