en

Intended for use

Tooth protector. A dental splint intended to protect teeth from damages caused by instrument in connection with peroral endoscopy, intubation or heavily biting in connection with ECT. **Used at hospitals by professional user.**

Contraindications

Pediatric patients

Features

- CAMO disposable splints can be used both in the upper and lower jaws.
- The size of the splint has been constructed to fit 90% of all jaw sizes. The size can, however, be easily adjusted with scissors.
- Both jaws can be protected simultaneously without excessively diminishing the necessary area for peroral endoscopy.
- The splints absorb high pressure without damaging teeth, thus the splints may be used during all kinds of pathological or induced excitations.
- Biocompatible material

In case of a serious incident from using the instruments, immediately contact Directa AB and the applicable authority in your country.

Precaution/ Warnings

- Do not reuse
- Shall not be sterilized or re-used as intended use can be jeopardized
- Film wrapping the filling shall not be removed as it prevents fragment dislodging.

Serious Incidents

In case of a serious incident from using the instruments, immediately contact Directa AB and the applicable authority in your country.

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DIRECTA AB