DIRECTA Camo[®]

en

Intended purpose

Disposable tooth protector. A dental splint intended to protect teeth from damages caused by instrument in connection with peroral endoscopy, intubation or heavily biting in connection with ECT.

Intended patient population

Patient with permanent teeth.

Intended users and environment

Professional users at hospitals.

Device description

Camo consists of a splint and filling wrapped by a thin plastic film, and a cotton cord attached to the splint.

Camo can be used both in the upper and lower jaws.

The splint can be adjusted with scissors to fit the jaw.

Both jaws can be protected simultaneously without excessively diminishing the necessary area for peroral endoscopy.

Indication

- Endoscopies.
- Intubation.
- ECT/Spasticity.

Contraindication

- Pediatric patients.
- Patients with fragile, loose teeth or insufficient dentition.

Warning

- Do not reuse due to cross-contamination. The device cannot be reprocessed due to the filling.
- Shall not be sterilized prior to use due to the properties of the materials.
- The film wrapping the filling must not be removed as it prevents fragment dislodging.
- Do not remove the filling, as the splint alone may not give sufficient protection.
- Materials have no x-ray contrast.
- Never use on edentulous patients, as retention cannot be achieved.
- Do not use if package is unintentionally opened.

Precautions

- If splint is adjusted, using a scissors, ensure that filling and the film is intact.
- Patient must have a sufficient number of teeth for adequate retention.
- Remove carefully — use both hands, not just the cord.

Recommended conditions of storage

Store at 15-23°C, temperatures beyond this range may affect material quality.

Shelf life

Shelf life is 3 years from manufacturing date.

Do not use after expiration date as the material properties cannot be guaranteed.

Expiration date on primary packaging and multiple package.

Safe disposal

Be aware of contaminated product. Use gloves.

Serious incident

In case of a serious incident in relation with the device, immediately report to Directa AB and the competent authority in the country where the serious incident occurred.

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DIRECTA AB