
# DIRECTA

# Sendoline®

## SendoCem MTA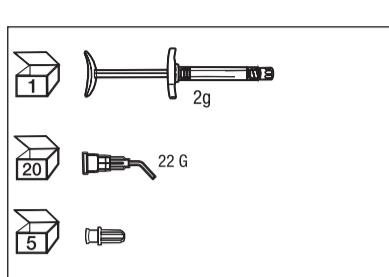

**Directa AB**  
**Finvids väg 8**  
**SE-194 47 Upplands Väsby, Sweden**  
**www.directadental.com**

**MARUCHI**  
**101, Donghwagongdanro, Munmak-eup,**  
**Wonju-si, Gangwon-do**  
**26365, Republic of Korea**  
**Tel: +82 33 734 0330**  
**Fax: +82 33 746 2804**

**EC** **REP**  
**KTR Europe GmbH**  
**Mergenthalerallee 77**  
**65760 Eschborn, Germany**  
**Tel: +49 6196 887 170**  
**Fax: +49 6196 887 1728**

107065-2026-01-29\_REV01

#### Product description

SendoCem MTA is a root canal filling material conforming to ISO 6876, in a pre-loaded syringe that does not require any pre-mixing and is set by absorbing moisture from the root canal environment. It is designed to prevent irritation of pulp and secondary infections.

#### Indication for use

Indirect pulp capping, direct pulp capping, pulpotomy, root end filling and Repair of perforation.

#### Intended use

Root canal filling/ capping and root perforation repair, effectively preventing irritation of pulp and secondary infections.

#### Intended user

The product is only to be used in a clinical or hospital environment by qualified dentists.

#### Application Field

The product is used for vital tooth therapy (pulp capping, Pulpotomy), root end filling, and Repair of perforation.

#### Patient population

All age group

This product is applicable for the group of patients who need endodontic treatment.

- Single use

#### Contraindication

Do not use in patients who are allergic to any of the product’s ingredients.

#### Combination with the device

There is no other devices to be used.

#### Composition

Zirconium dioxide, Calcium silicate, Calcium aluminate, Calcium sulfate, Dimethyl sulfoxide, Lithium carbonate and Thickener

#### Performance Characteristics (According to ISO 6876)

- Setting time : 3-5 min
- Radiopacity : The same radiopacity as that of aluminum of above 3 mm in thickness
- Solubility : below 3.0%

#### Precaution for use

- Do not use in patients with allergic reaction to chemicals (specifically to Dimethyl Sulfoxide).
- For dentists use only.
- Make sure that the product does not pass over to the mandibular canal or maxillary sinus.
- Do not use large amounts to the root canal because it is difficult to remove

#### General precautions

- Check the product and packaging for abnormal condition or damage before use.
- Do not use other than for intended use.
- Keep in pouch provided before use to prevent a potential adverse reaction by carbon dioxide and moisture present in the atmosphere.
- Do not contact unhardened product with the skin or mucous membrane. If contact occurs, rinse with large amounts of water immediately.
- Wear protective gloves, mask and glasses.
- The product must be stored at room temperature avoiding high temperature and humidity

#### Application precautions

- Make sure that the product comes out smoothly through the dedicated tip.
- Wash the treatment area with distilled water before application.
- The product shall be applied with a thickness of at least 3 mm.
- The product should be used immediately after replacing the dedicated tip because it can harden when in contact with air.
- Replace the dedicated tip immediately after use with a new one keeping the tip to minimize the product from contamination and infection during storage. Extrude a small amount of product exposed to the air prior to store. Remove the old dedicated tip prior to use in another application and replace with a new dedicated tip.
- To reduce the risk of discoloration of the teeth, it should be applied carefully to aesthetic areas to avoid exposure to saliva.

#### Adverse reactions

No adverse reaction has been reported to date.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### Step by step instruction

##### PREPARATION PRIOR TO USE

- Check the product and packaging for abnormal condition or damage before use.
- The dedicated tip should be changed with a new one prior to use. Make sure that the product comes out smoothly

##### Direction for use

- Indirect pulp capping
  - Isolate the tooth with rubber dam and form cavity prep using a high speed bur.
  - Place a new dedicated tip on the syringe, check if the product can be extruded smoothly.
  - The product is applied to the deep dentin close to the pulp chamber, and distilled water is applied.
  - After 3 minutes, remove moisture with a sterile cotton pellet, and gently pack it to avoid dead space.
  - Make temporary restoration with self-curing glass ionomer etc.
  - Expose radiographs for verification of achievements.

##### 2) Direct pulp capping

- Isolate the tooth with rubber dam and form cavity prep using a high speed bur.
- When the pulp horns is exposed, it is sufficiently disinfected with undiluted NaOCl.
- Wash out with distilled water.
- Place a new dedicated tip on the syringe, check if the product can be extruded smoothly.
- Apply the product to the exposed pulp horns and control bleeding by pressing it with a cotton pellet.
- Wash out the blood with distilled water.
- If necessary, the product is applied additionally with a thickness of 3mm or more.
- Apply distilled water on the product and wait for 3 minutes.
- Remove moisture with a sterile cotton pellet, and gently pack it to avoid dead space.
- Make temporary restoration with self-curing glass ionomer etc.
- Expose radiographs for verification of achievements.

##### 3) Pulpotomy

- Isolate the tooth with rubber dam and form cavity prep using a high speed bur.
- Remove pulp horns to the appropriate depth.
- Disinfect the surface of pulp horns with undiluted NaOCl, etc.
- Rinse the pulp horns and cavity disinfected with distilled water.
- Place a new dedicated tip on the syringe, check if the product can be extruded smoothly.
- The product is applied to the pulp horns and the surrounding hard tissue.
- Apply distilled water on the product and gently pack it with a wet cotton.
- Wash out the blood with distilled water.
- The product is applied additionally with a thickness of 3 mm.
- Apply distilled water on the product and wait for 3 minutes.
- Pack the product, dry it well, and make temporary restoration with glass ionomer etc.
- Expose radiographs for verification of achievements.

##### 4) Repair of perforation

- Isolate the tooth with rubber dam and disinfect the treatment area.
- Dry the treatment area.
- Place a new dedicated tip on the syringe, check if the product can be extruded smoothly.
- Apply the product to the treatment area.
- Expose radiographs for verification of achievements. If the perforated site does not get filled completely, pack with an appropriate instrument, and after adding the product, a radiograph is taken.
- Perform endodontic treatment.

##### 5) Root-end filling

- After approaching the apical area, resect apical 3 mm with surgical bur.
- Form a 3 mm deep cavity on the resected apex.
- The bleeding area is compressed with a cotton pellet, etc. along with hemostatic agent.
- Discolor with methylene blue to check for cracks or accessory canals in the apical region and observe with microscope.
- Place a new dedicated tip on the syringe, check if the product can be extruded smoothly.
- Wash the reverse filling cavity with distilled water and apply the product.
- After applying the wet cotton to the product, wait 3 minutes.
- Pack the product with and appropriate instrument. If necessary, apply additional products.
- Expose radiographs for verification of achievements.

##### Instruction for storage

- Store at room temperature avoiding high temperature and humidity.
- A new dedicated tip replaces it in next use.
- After use do not recap the used dedicated tip, twist the storage cap provided with the product tightly onto the syringe head.
- Once the gray cap, which was initially fastened to the syringe, is separated, it will not be sealed even if it is fastened again. It must be sealed using the enclosed storage cap.
- Place syringe with the storage cap in the provided pouch. Seal the pouch completely.
- Tighten the storage cap immediately after use to minimize the product from contamination and infection during storage.

##### Shelf life

2 years from date of manufacture.

##### Disposal

Adhere to national laws and recommendations of local authorities for proper disposal.

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