# DIRECTA CONTACTEZ®

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**Composition**

Plastic (polycarbonate)

Diamond plated and non-plated stainless steel

**Precaution**

- To be cleaned and disinfected according to the instructions below prior to first use and between uses. If high-level disinfection not possible or sterilization required by national recommendations, the device shall be sterilized according to the instructions.
- Use minimal vertical force to avoid breakage.

**User instructions**

Use two fingers and carefully pass the strip buccolingually, back and forth with gentle force.

**Examples of use**

*ContactEZ Restorative Strip System:* Proximal contact adjustment and complete marginal seating of crowns, veneers, inlays/onlays as well as proximal contouring of composite fillings.

*ContactEZ IPR Strip System:* Interproximal reduction system.

*ContactEZ Subgingival Strip System:* Removal of hard-to-reach subgingival rough margins and overhangs as well as final polishing.

*ContactEZ Hygienist Strip:* Removal of extrinsic discoloration and plaque from interproximal surfaces.

## Maintenance, cleaning and sterilization

Product	<b>ContactEZ Strips</b> are used to remove and/or finish dental enamel or filling materials
Warnings	Place in the middle of the autoclave, not against the autoclave's walls or heating element
Limitations on reprocessing	The characteristics of the plastic are affected by autoclaving, resulting in limited product lifetime. Metal strip may discolor after autoclaving, does not affect safety or performance. Detergent containing amines and cationic surfactants in combination with sterilization in autoclave may shorten lifetime of the product/strips.
Inspection and maintenance	All plastic products have a limited-service lifetime and should be replaced at regular intervals. All types of cleaning reduce the product's service lifetime. The product should be replaced when its functioning and/or performance no longer satisfies the specifications in this manual.

## Instructions

Preparation before cleaning	Instruction	Wipe off the instrument after use to prevent soil and debris to dry onto the instrument. Perform cleaning as soon as possible after use. Do not exceed 2 hours.
Cleaning: Manual	Equipment	Soft bristle brushes of various sizes, Ultrasonic bath
	Detergent	Follow agent manufacturers recommendations for concentration and temperature. Manual: Enzymatic or low-alkaline (pH≤8) detergent suitable for manual cleaning. Ultrasonic bath: Enzymatic or low-alkaline detergent with minimal foaming characteristics.
	Water quality	Drinking water quality for manual cleaning and distilled or de-mineralized water for ultrasonic bath and final rinse should be used.
	Instruction	<ol style="list-style-type: none"> <li>1. Immerse in freshly prepared cleaning solution as per manufacturer's instructions although maximum 40°C. Clean mechanically with a brush, working beneath the liquid level, until visibly clean.</li> <li>2. Rinse thoroughly</li> <li>3. Clean in an ultrasonic bath for a minimum of 5 minutes, using a frequency of 35-45 KHz and power min. 150 W.</li> <li>4. Rinse thoroughly for min 30 seconds.</li> <li>4 a Carefully dry instruments with clean compressed air (Class 1 or better according ISO 8573-1:2010) or lint free wipe. Any remaining water can cause potential rust or corrosion on the instrument surface and into the substrate.</li> <li>5. Inspect cleanliness. If necessary, repeat from step 1.</li> </ol>
Disinfection: Manual	Detergent	Immerse into suitable disinfection solution. Follow the manufacturer's instruction of disinfection solution for concentration and time. Disinfection in alcohol is recommended for best product lifetime.

Cleaning and disinfection: Automated	Equipment Detergent	Washer-disinfector (validated according to EN ISO 15883) Enzymatic or low alkaline, suited for medical devices. Non-corrosive, neutral cleaning agent with minimal foaming characteristics, suited for medical devices.
	Rinsing Agent	Follow agent manufacturer's instructions regarding concentration and temperature.
	Water quality	Drinking water quality for cleaning and deionized or purified water for final rinse/ disinfection.
	Instructions	<ol style="list-style-type: none"> <li>1. Load the instruments in the washer-disinfector. Use a suitable instrument tray.</li> <li>2. Run program: <ul style="list-style-type: none"> <li>- Pre-wash in cold water, 2 x 2 min</li> <li>- Main wash with detergent at minimum 55°C, 10 min</li> <li>- Rinse in warm water, 2x 1 min.</li> <li>- Final rinse/disinfection in distilled or de-mineralized water at 90°C, minimum 1 min.</li> <li>- Drying at 110° C, minimum 15 min</li> </ul> </li> <li>3. When unloading, control that the instruments are clean. If necessary, repeat from step 1 or use manual cleaning</li> </ol>

Packaging for sterilization	Equipment Instruction	Standard packaging pouch  Pack the instrument in the pouch. Place pouches plastic towards plastic and paper towards paper.
Sterilization	Caution  Equipment Instructions	The instrument must be cleaned and disinfected before sterilization.  Steam autoclave (validated according to EN 13060, EN 285, EN ISO 17665)  Run minimal cycle: Steam temperature Min. 134 °C / 273°F Steam exposure time 4 min Vacuum drying 3 min
Storage	N/A	
Transportation	N/A	

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**DIRECTA AB**